


# TIMS MVP™ Workflow Card

Worklist

or Manual

1. From the Study Dashboard's Worklist tab, double-click on your schedule study.
2. Or click NEW BLANK STUDY and enter patient data.
3. Click **OK**.
4. Capture streaming video or images.
5. Review the study (optionally, edit and/or label the study)
  - a. Send the study to PACS/TIMS Review
  - b. Archive the study to CD/DVD/USB/Network

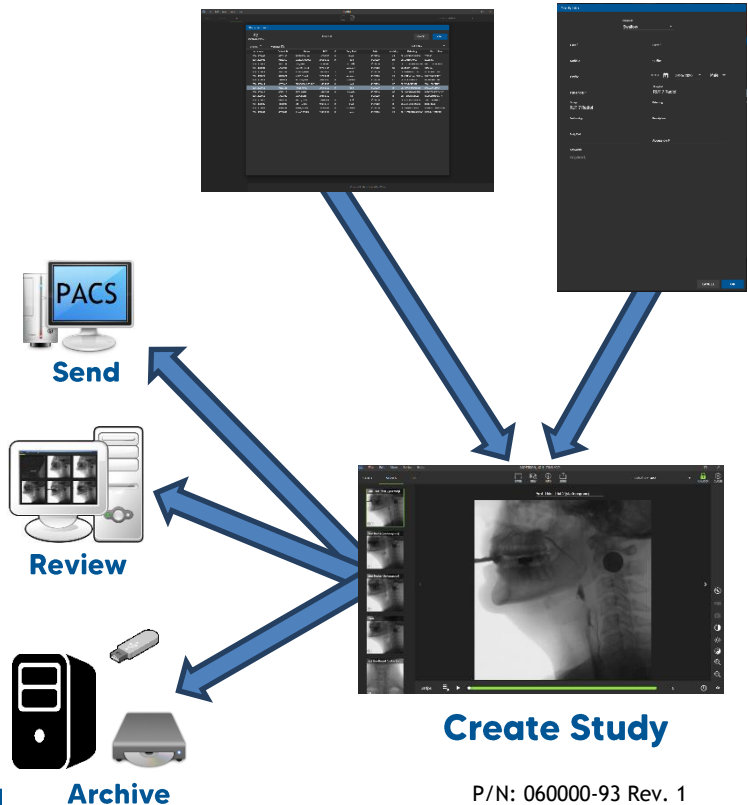

**TIMS**  
MEDICAL

## TIMS MVP records & stores individual procedures as Studies




### Studies are created & managed in the Study Dashboard

- The STUDIES tab lists recorded Studies
- The WORKLIST tab displays today's scheduled procedures



### Three main tabs are available while recording a study

- The LIVE tab displays the Live Source for Recording
- The STUDY tab displays an overview of the study with thumbnails of each Series.
- The SERIES tab provides for detailed view, editing & analysis of each Series

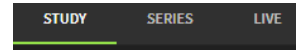

### TIMS MVP auto-saves all recording and edit

- There is no save command, all recording & changes are saved automatically

### TIMS MVP uses non-destructive editing

- Series & image deletes do not affect any underlying image data
- Everything can be restored to the original state at any time

### TIMS MVP is modeless

- Move to any tab or window at any time