
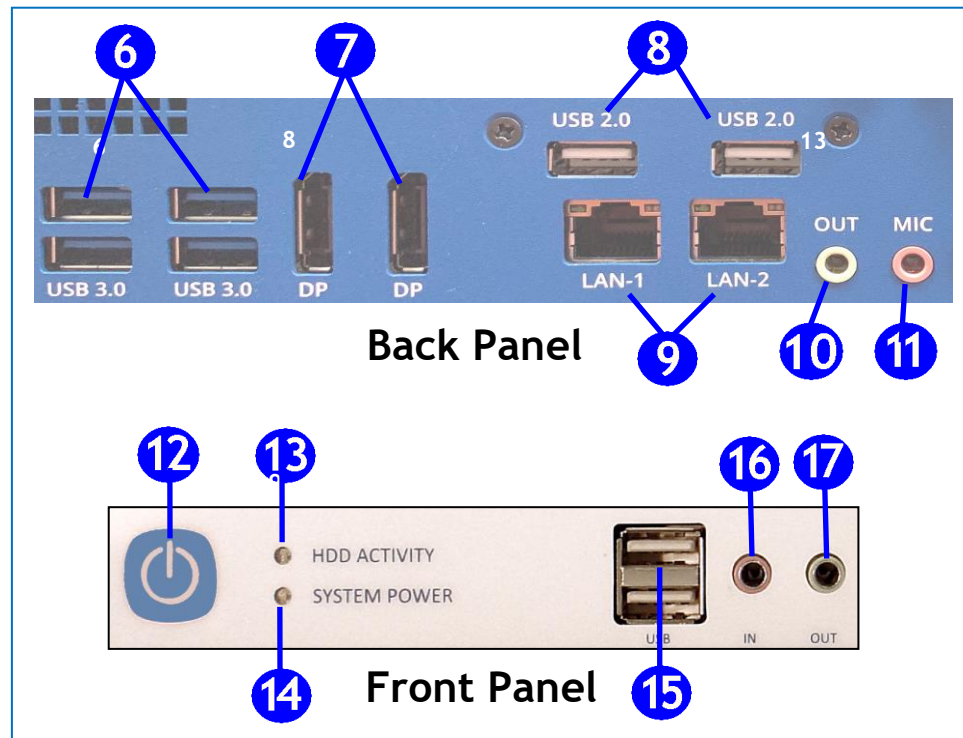


TIMS MVP® Setup and Shortcuts Card

GET CONNECTED

MODALITY CONNECTION HARDWARE

- 1 Trigger Adapter
- 2 DVI to BNC Cable or DVI to BNC Adaptor or DVI to DVI Cable (optional)
- 3 DVI-D to HDMI cable (optional) or HDMI to HDMI cable (optional)
- 4 S-video to S-video cable or for composite color an S-video to composite color adaptor (optional)

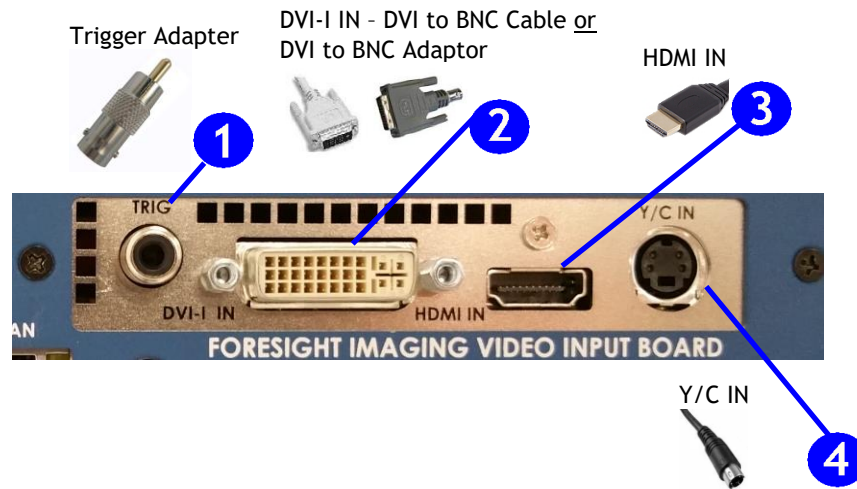

MODALITY CONNECTIONS

- 1 Trigger In
- 2 RGB, DVI, or Monochrome In
- 3 HDMI In
- 4 Y/C In: S-Video or Composite Color NTSC or PAL

PC CONNECTIONS

- 5 External Power Supply
- 6 USB 3.0 for higher speed DVD drives, keyboard and mouse
- 7 DisplayPort Out- right DP is the primary
- 8 USB 2.0 for keyboard and mouse
- 9 LAN-1 for network; LAN-2 is not typically used - available for accessory devices
- 10 Speakers Output
- 11 Microphone In
- 12 Standby Switch
- 13 Hard Drive Activity LED
- 14 Power On LED
- 15 USB for keyboard and mouse
- 16 Microphone In
- 17 Speakers Output



TIMS MVP Shortcut Keys - Sorted by Function

Shortcut	Action
Capture	
F12	Snap single image frame
SPACEBAR	Start/Stop record
Editing	
DELETE	Delete selected study or image
CTRL + ←	Move to first frame of series
CTRL + →	Move to last frame of series
I	Set trim start
I	Set trim end
CTRL + T	Split a series view into two views
CTRL + U	Combine series view with the next view
CTRL + D	Delete video or image snapshot
TAB / SHIFT + TAB	Next or previous label
Viewing	
CTRL + 1	View the Study tab
CTRL + 2	View the Series tab
CTRL + 3	View the Live tab
CTRL + =	Enlarge/Zoom in
CTRL + -	Shrink/Zoom out
SPACEBAR	Start/Stop play

Shortcut	Action
Operation	
CTRL + O	Open the Study Dashboard to open studies
SHIFT + I	Open the Study Info dialog
CTRL + N	Create a new blank study
CTRL + F4	Close study
CTRL + L	Loop video play mode
SHIFT + ←	Skip backward 10 frames
SHIFT + →	Skip forward 10 frames
CTRL + SPACEBAR	Toggle play direction
HOME	Move to start
END	Move to end
CTRL + ALT + C	Copies study info to clipboard
CTRL + C	Copy image to clipboard
CTRL + SHIFT + C	Copy image with Area of Interest
CTRL + F	Flag/Unflag a series to send
CTRL + I	Set timing point
CTRL + I	Clear timing point
ALT + F4	Quit TIMS MVP
Navigation	
↑ ↓	Move series to series
← →	Move frame to frame